Contact Us

TEL : (02)2395-7378

FAX : (02)2395-7379

Mon. - Fri. 9:00am - 6:00pm

service@qtmedical.com

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QT Cloud

(for training purpose)

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<https://dashboard.qtmedical.com/login>

Web service version:
V1.15.4 (新加坡)
V1.15.5 (其他地區)

Version: V2-R6

Update Date: 09/13/2022

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A screenshot of the QT Medical Organization Activation form. The form is titled 'Organization Activation' and has a dark purple header with the QT Medical logo and 'Login' and 'Activate' links. The form contains several sections: 'Fields marked with an asterisk (*) are required.', 'PCA 500 Recorder Serial Number (How to Identify?)', 'Organization Name', 'Facility Registration Number', 'Country' (a dropdown menu with 'Australia' selected), 'Address', 'Phone Number', 'Admin Account Information' (which includes fields for 'Email Address', 'First Name', 'Middle Initial', and 'Last Name'), and 'Fill in Your Physician Registration Number To Grant Physician Permission.'. The 'Admin Account Information' section is circled in pink. At the bottom left, there is a checkbox labeled 'I have read and accepted the Terms of Service and Privacy Policy.' and a pink arrow points to the 'Submit' button next to it. A 'Reset' button is located at the bottom right.

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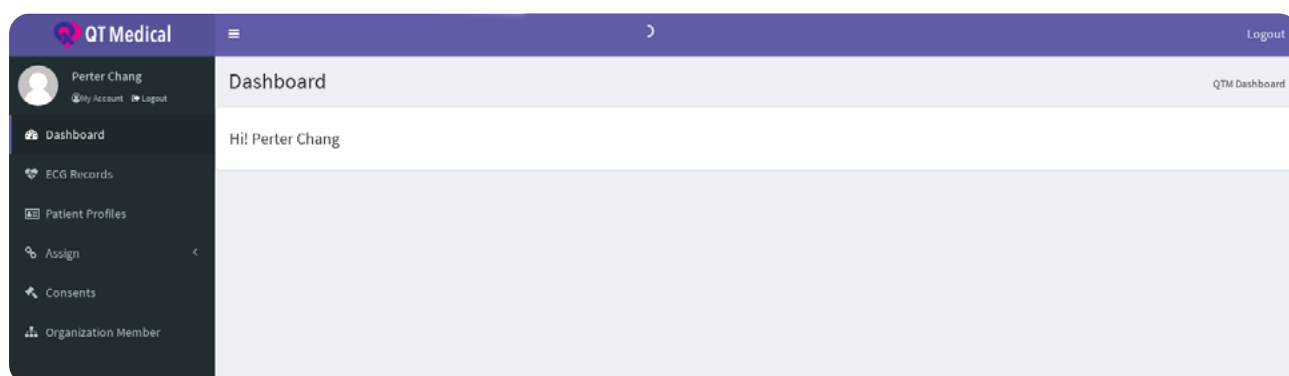
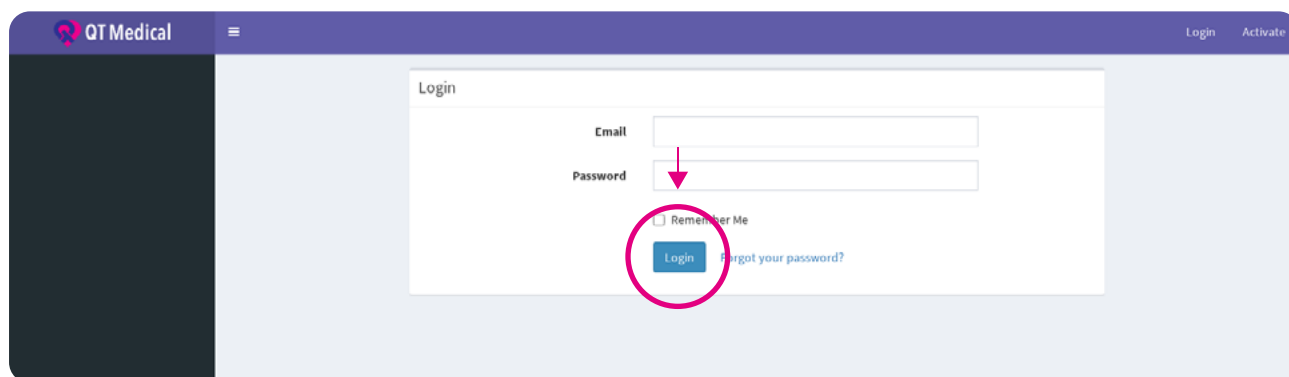
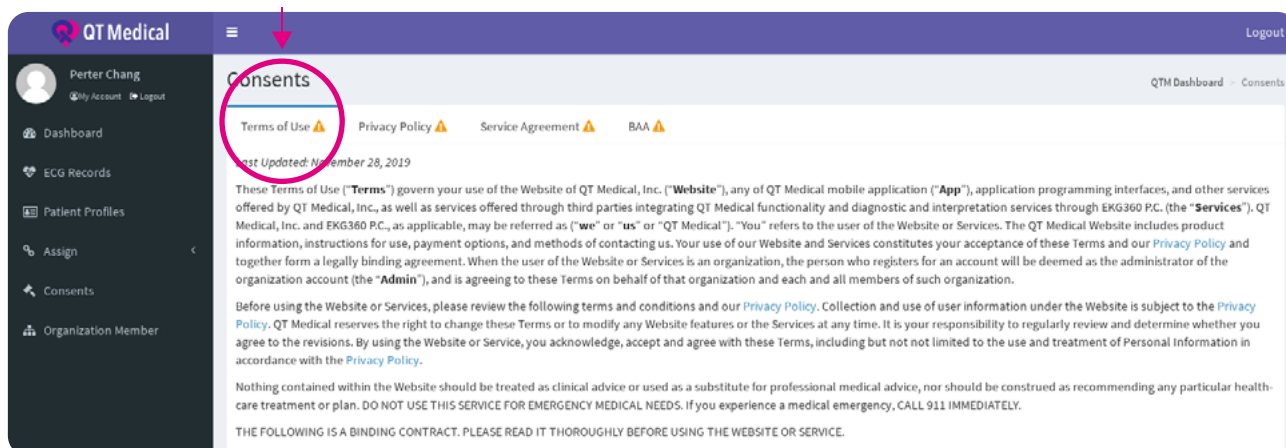
You Can Contact Us About This Policy and Your Information.

If you have any questions about this Privacy Policy or our data practices, please email us at privacy@qtmedical.com. You can also write or call us at:

QT Medical, Inc.
Attn: Brett Chien, PhD
1360 Valley Vista Dr., Suite 203, Diamond Bar, CA 91765
(909)323-0007

We will not take action against you for filing a complaint. If you have a complaint concerning our compliance with applicable privacy laws, we will investigate your complaint and take appropriate measures. You may also file a complaint relating to our use and disclosure of your PHI to the United States Department of Health and Human Services Office for Civil Rights at <https://ocrportal.hhs.gov/ocr/smartscreen/main.jsf>.

We May Update This Policy.

This Policy may be updated from time to time without further notice to you and this will be reflected by a "Last modified" date above. Please revisit this webpage regularly for any changes. By continuing to use the Services, you are consenting to the terms of the then-current Policy.

Agree

Copyright © 2013- 2020 QT Medical, Inc. All rights reserved.

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QT Medical Logout

Perter Chang
My Account Logout

Dashboard
ECG Records
Patient Profiles
Assign
Consents
Organization Member

Consents QTM Dashboard > Consents

Terms of Use Privacy Policy Service Agreement BAA

Last Updated: November 28, 2019

These Terms of Use ("Terms") govern your use of the Website of QT Medical, Inc. ("Website"), any of QT Medical mobile application ("App"), application programming interfaces, and other services offered by QT Medical, Inc., as well as services offered through third parties integrating QT Medical functionality and diagnostic and interpretation services through EKG360 P.C. (the "Services"). QT Medical, Inc. and EKG360 P.C., as applicable, may be referred as ("we" or "us" or "QT Medical"). "You" refers to the user of the Website or Services. The QT Medical Website includes product information, instructions for use, payment options, and methods of contacting us. Your use of our Website and Services constitutes your acceptance of these Terms and our Privacy Policy and together form a legally binding agreement. When the user of the Website or Services is an organization, the person who registers for an account will be deemed as the administrator of the organization account (the "Admin"), and is agreeing to these Terms on behalf of that organization and each and all members of such organization.

Before using the Website or Services, please review the following terms and conditions and our Privacy Policy. Collection and use of user information under the Website is subject to the Privacy Policy. QT Medical reserves the right to change these Terms or to modify any Website features or the Services at any time. It is your responsibility to regularly review and determine whether you agree to the revisions. By using the Website or Service, you acknowledge, accept and agree with these Terms, including but not limited to the use and treatment of Personal Information in accordance with the Privacy Policy.

Nothing contained within the Website should be treated as clinical advice or used as a substitute for professional medical advice, nor should be construed as recommending any particular health-care treatment or plan. DO NOT USE THIS SERVICE FOR EMERGENCY MEDICAL NEEDS. If you experience a medical emergency, CALL 911 IMMEDIATELY.

THE FOLLOWING IS A BINDING CONTRACT. PLEASE READ IT THOROUGHLY BEFORE USING THE WEBSITE OR SERVICE.

QT Medical Logout

Perter Chang
My Account Logout

Dashboard
ECG Records
Patient Profiles
Assign
Consents

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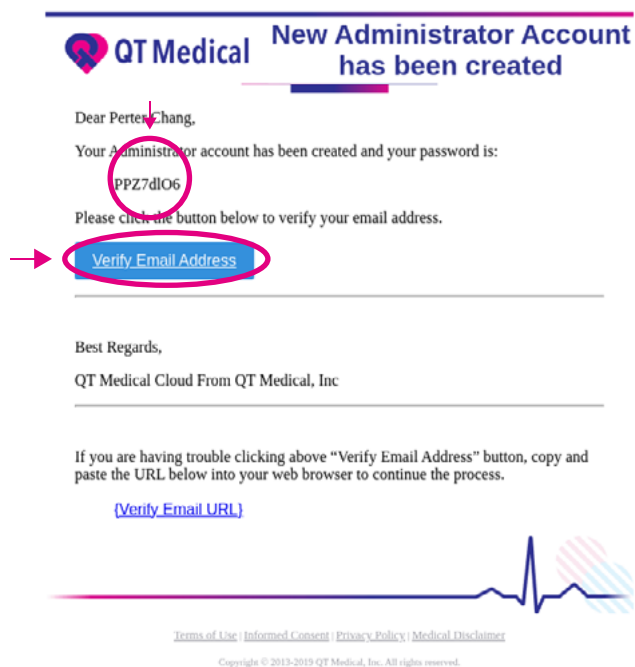
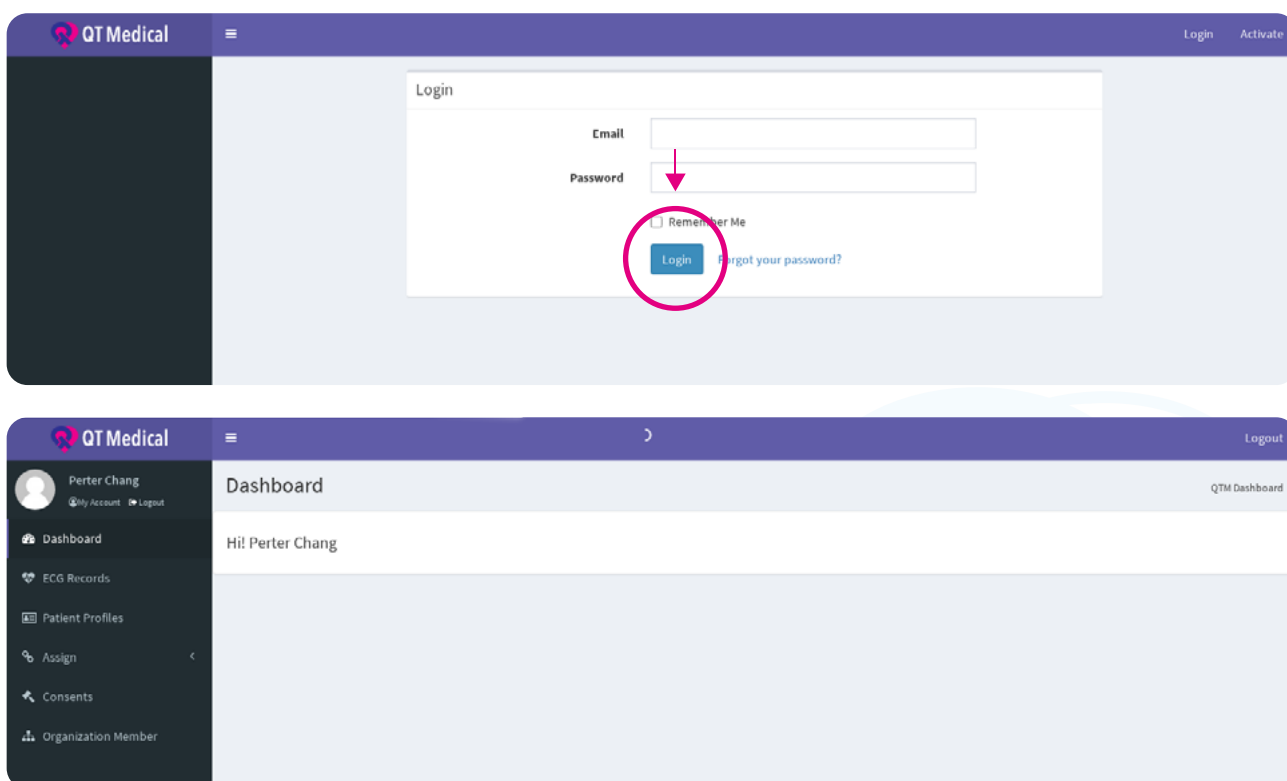
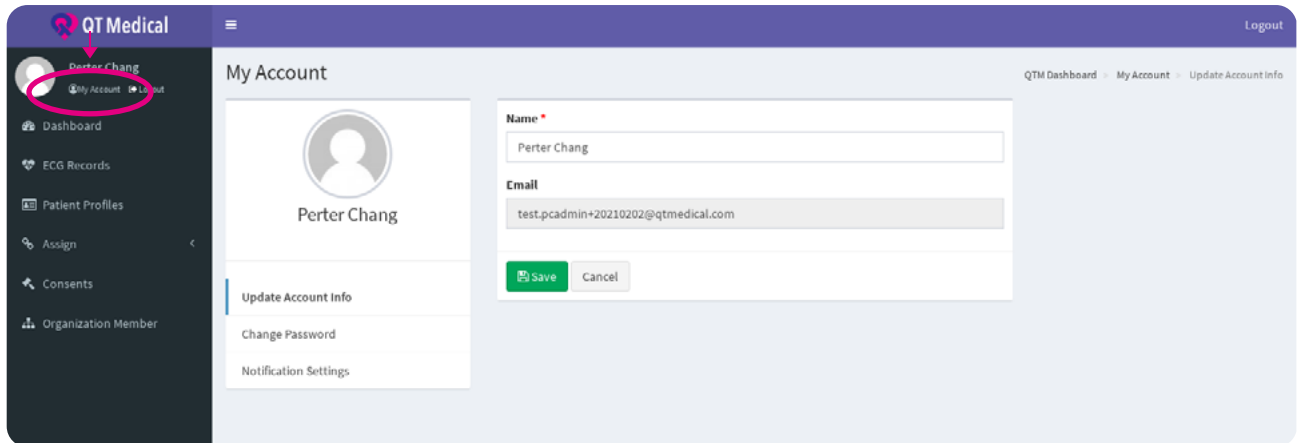
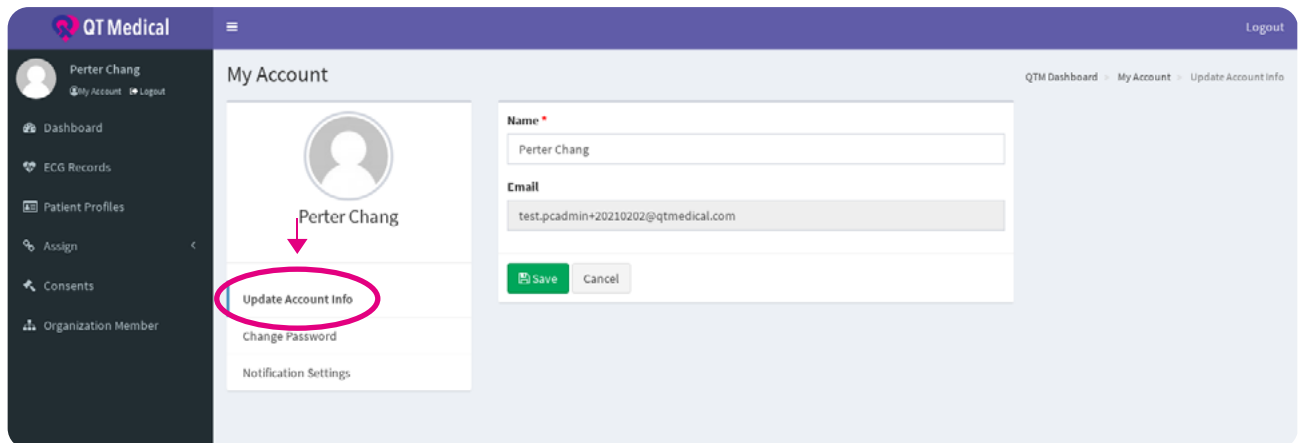
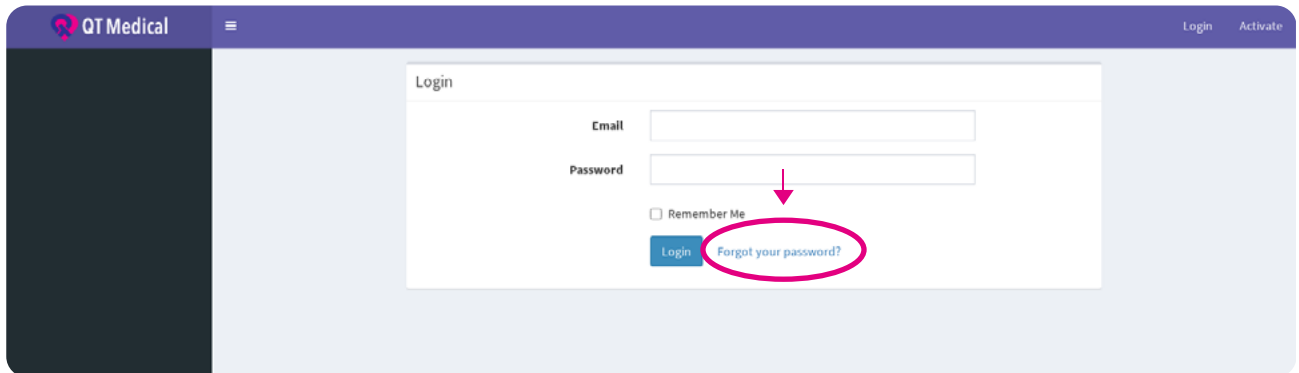
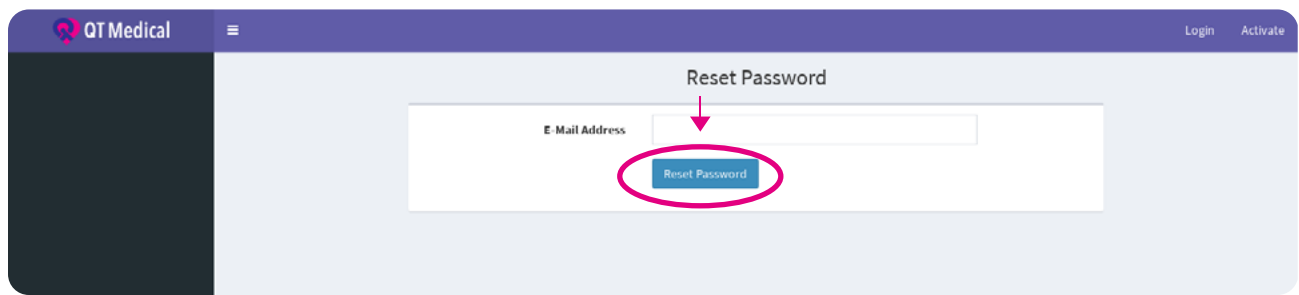
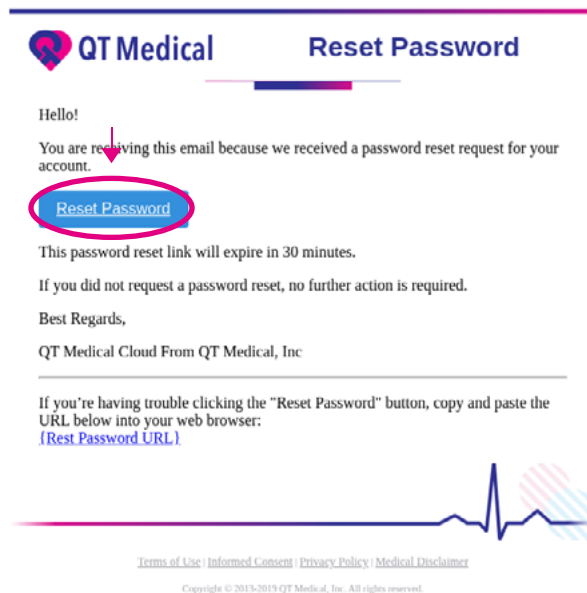
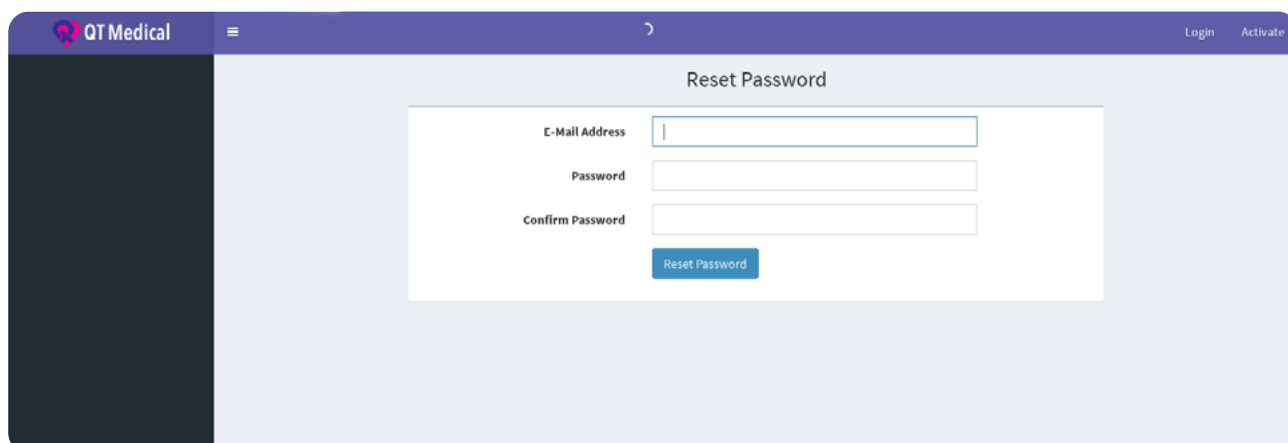
The screenshot shows a web application interface for "QT Medical". The header is dark blue with the logo on the left and "Login" and "Activate" links on the right. A dark blue sidebar is on the left. The main content area is light blue and contains a white "Reset Password" form. The form has three input fields: "E-Mail Address", "Password", and "Confirm Password". Below the fields is a blue "Reset Password" button.

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The screenshot shows the 'My Account' page for a user named Jon Snow. On the left sidebar, the 'Change Password' option is circled in pink. The main content area displays the 'Change Password' form with three input fields: 'Old Password', 'New Password', and 'Confirm Password'. Below these fields is a green 'Change Password' button and a grey 'Cancel' button. The breadcrumb trail at the top right reads 'QTM Dashboard > My Account > Change Password'.

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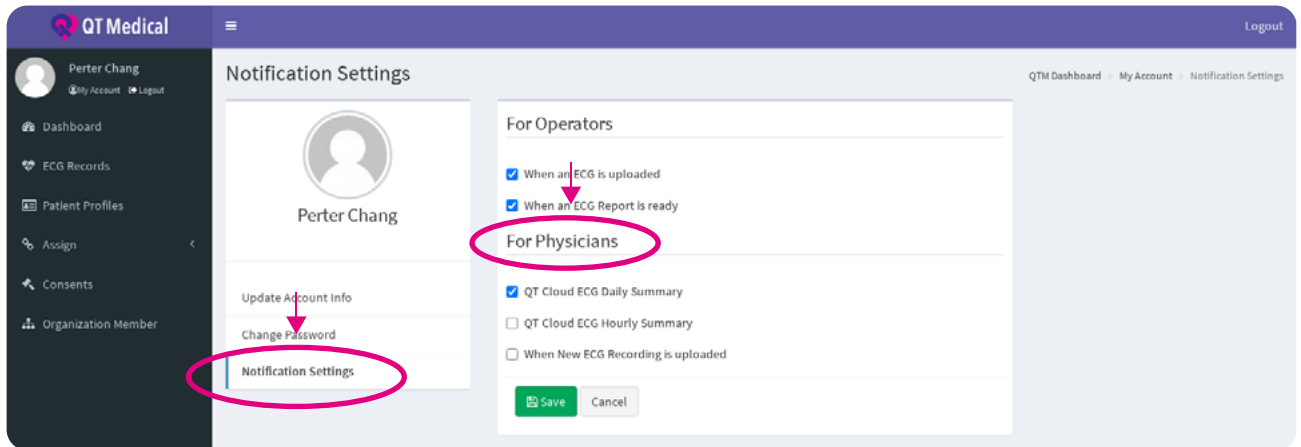
This screenshot shows the same 'My Account' page, but now the 'Change Password' button in the main content area is circled in pink. The breadcrumb trail at the top right reads 'QTM Dashboard > My Account > Change Password'.

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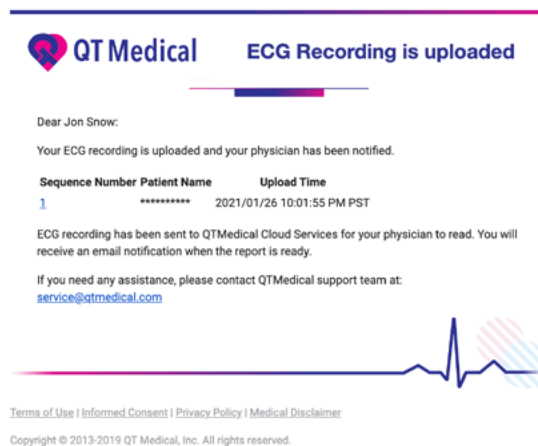
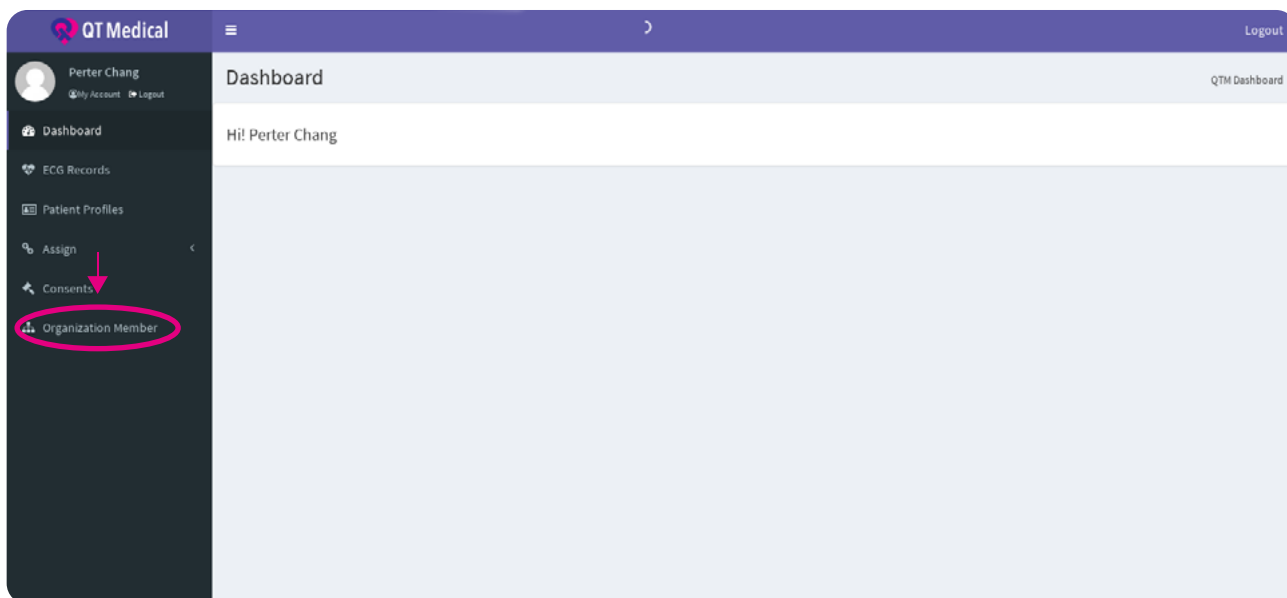
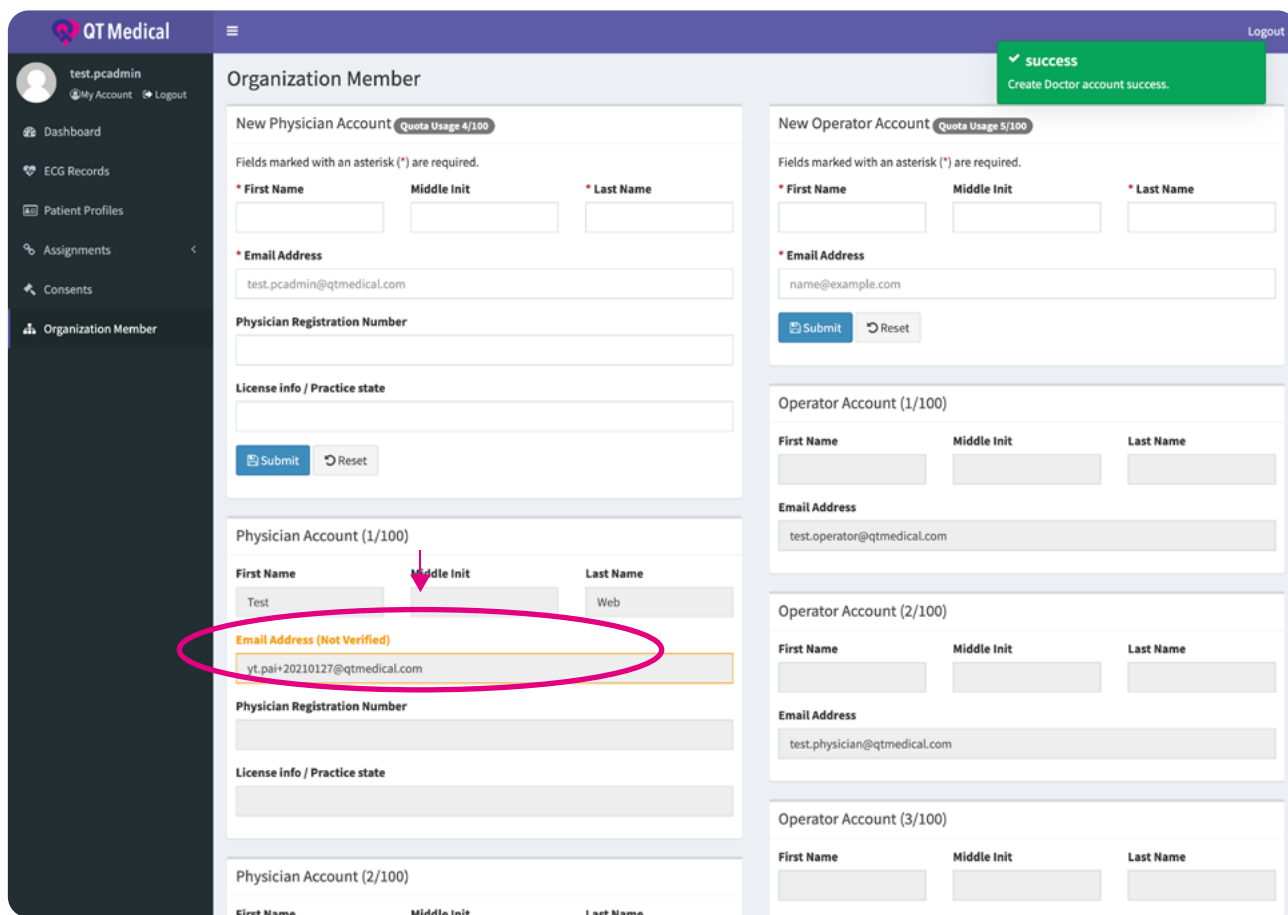
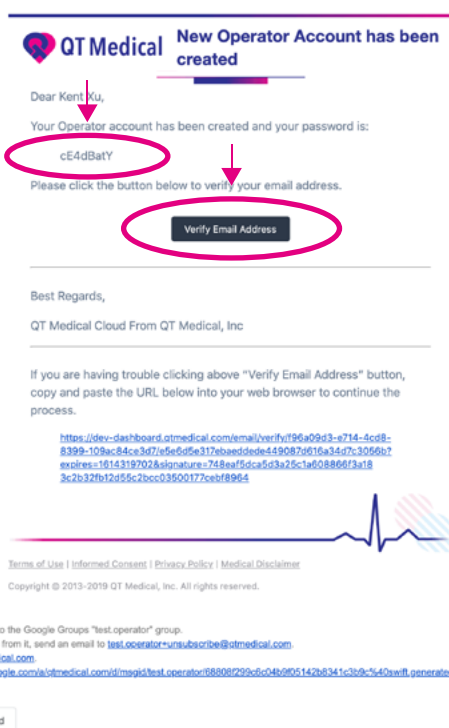
A screenshot of the 'Organization Member' form in the QT Medical system. The form is divided into two main sections: 'New Physician Account' and 'New Operator Account'. Both sections have a 'Quota Usage' indicator (3/100 for Physician, 5/100 for Operator). Each section contains fields for First Name, Middle Init, Last Name, and Email Address, with a note that fields marked with an asterisk (*) are required. Below these fields are 'Submit' and 'Reset' buttons. In the 'New Physician Account' section, the 'Submit' button is circled in red, with a red arrow pointing to it and the text '建立新的醫師帳號' (Create new physician account). In the 'New Operator Account' section, the 'Submit' button is also circled in red, with a red arrow pointing to it and the text '建立新的技術師帳號' (Create new technician account). Below these sections are lists for existing 'Physician Account (1/100)' and 'Operator Account (1/100)'.

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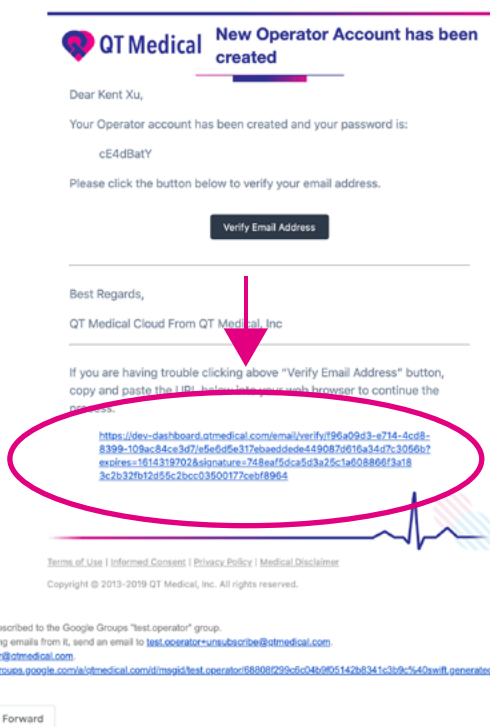
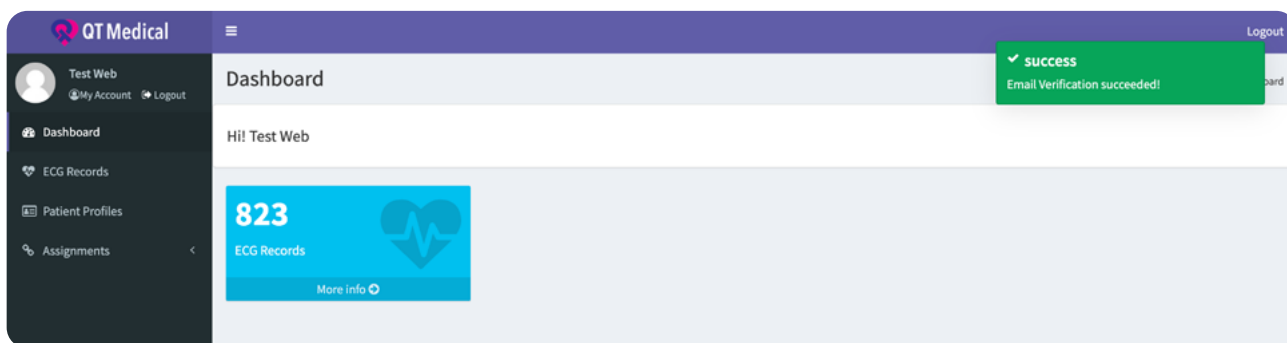
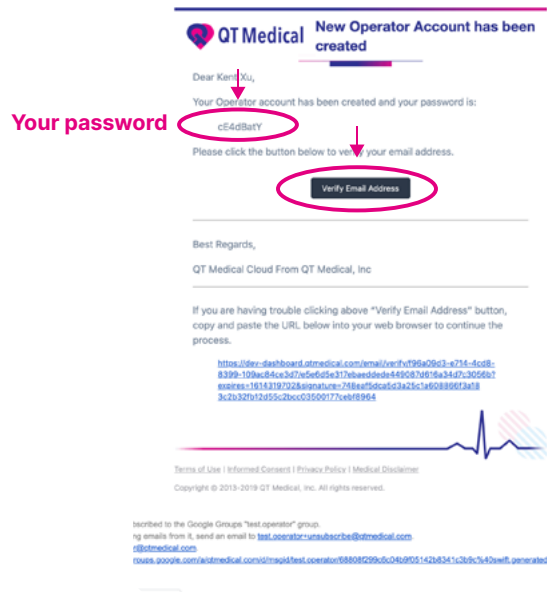
The screenshot shows the 'Organization Member' page in the QT Medical system. It features two main forms: 'New Physician Account' and 'New Operator Account'. Both forms have fields for First Name, Middle Init, Last Name, and Email Address. The 'New Physician Account' form also includes a 'Physician Registration Number' field and a 'License info / Practice state' field. The 'New Operator Account' form includes an 'Operator Account (1/100)' section with fields for First Name, Middle Init, Last Name, and Email Address. The 'Submit' button on the 'New Physician Account' form is circled in pink with a red arrow pointing to it, and the text '建立新的醫師帳號' (Create new physician account) is written next to it. The 'Submit' button on the 'New Operator Account' form is also circled in pink with a red arrow pointing to it, and the text '建立新的技術師帳號' (Create new technician account) is written next to it.

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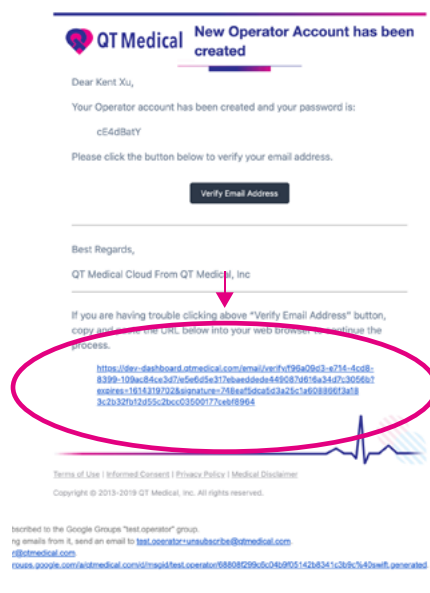
The screenshot shows the 'Organization Member' page in the QT Medical system after the 'New Operator Account' form has been submitted. A green success message 'success Create Operator account success.' is displayed at the top right. The 'New Operator Account' form is now filled out with the following data: First Name: name, Middle Init: , Last Name: , Email Address: name@example.com. Below this, the 'Operator Account (2/100)' section is shown with the following data: First Name: Kent, Middle Init: Xu, Last Name: Xu. The 'Email Address (Not Verified)' field in the 'Operator Account (2/100)' section is circled in pink with a red arrow pointing to it, and the text '建立新的技術師帳號' (Create new technician account) is written next to it. The 'Email Address' field in the 'Operator Account (2/100)' section is also circled in pink with a red arrow pointing to it.

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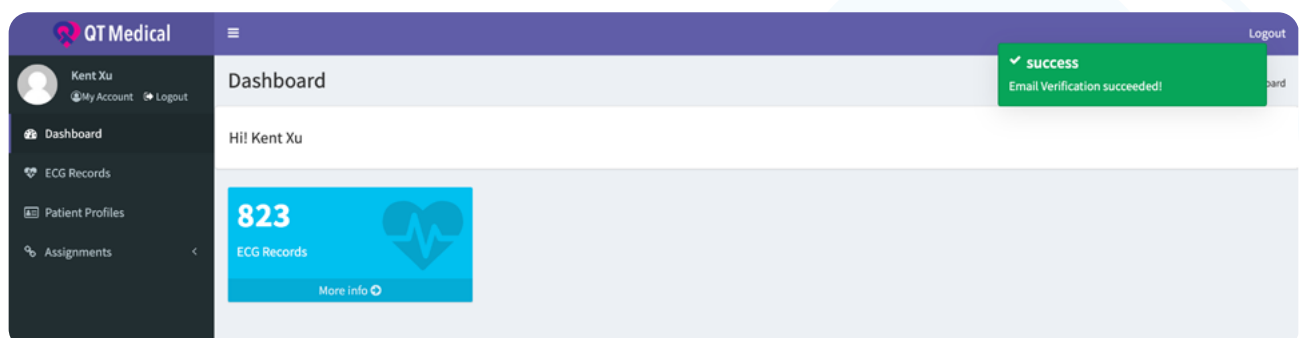
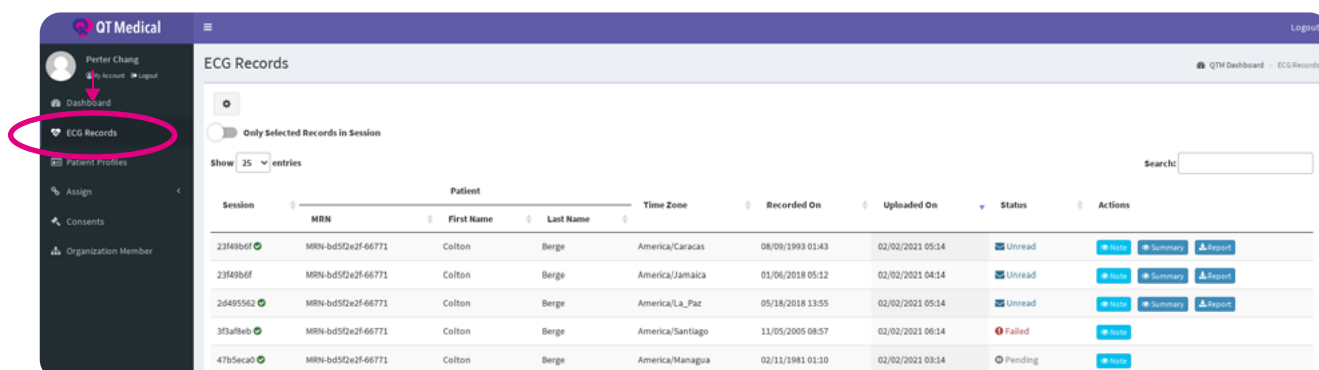
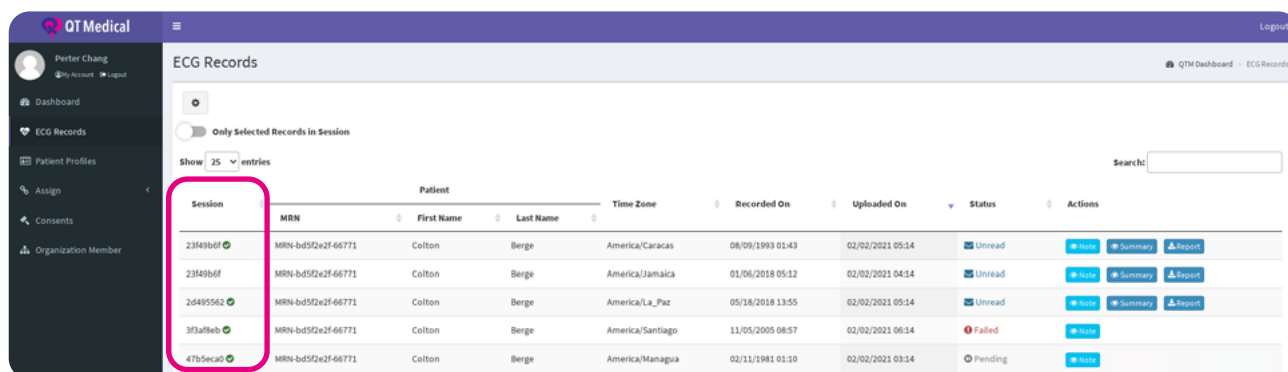
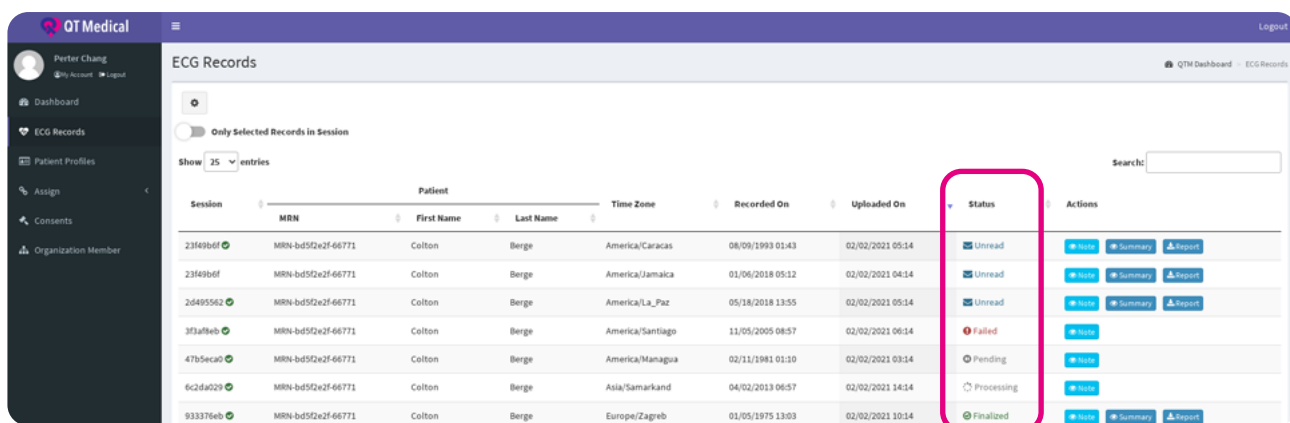
Organization Member

ECG Records

Only Selected Records in Session **Selected**

Show 25 entries

Search

Session	MRN	Patient	Time Zone	Recorded On	Uploaded On	Status	Actions
		First Name	Last Name				
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Caracas	08/09/1993 01:43	02/02/2021 05:14	Unread Note Summary Report
2d495562	MRN-bd5f2e2f-66771	Colton	Berge	America/La_Paz	05/18/2018 13:55	02/02/2021 05:14	Unread Note Summary Report
3f3a9beb	MRN-bd5f2e2f-66771	Colton	Berge	America/Santiago	11/05/2005 08:57	02/02/2021 06:14	Failed Note
47b5eca0	MRN-bd5f2e2f-66771	Colton	Berge	America/Managua	02/11/1981 01:30	02/02/2021 03:14	Pending Note
6c2da029	MRN-bd5f2e2f-66771	Colton	Berge	Asia/Samarkand	04/02/2013 06:57	02/02/2021 14:14	Processing Note

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Show 25 entries

Search

Session	MRN	Patient	Time Zone	Recorded On	Uploaded On	Status	Actions
		First Name	Last Name				
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Caracas	08/09/1993 01:43	02/02/2021 05:14	Unread Note Summary Report
2d495562	MRN-bd5f2e2f-66771	Colton	Berge	America/La_Paz	05/18/2018 13:55	02/02/2021 05:14	Unread Note Summary Report
3f3a9beb	MRN-bd5f2e2f-66771	Colton	Berge	America/Santiago	11/05/2005 08:57	02/02/2021 06:14	Failed Note
47b5eca0	MRN-bd5f2e2f-66771	Colton	Berge	America/Managua	02/11/1981 01:30	02/02/2021 03:14	Pending Note
6c2da029	MRN-bd5f2e2f-66771	Colton	Berge	Asia/Samarkand	04/02/2013 06:57	02/02/2021 14:14	Processing Note

QT Medical

Perter Chang

Dashboard

ECG Records

Patient Profiles

Assign

Consents

Organization Member

ECG Records

Only Selected Records in Session

Show 25 entries

Search

Note for ECG ID: a78aa272-695f-4b46-92c0-2e88486a332f

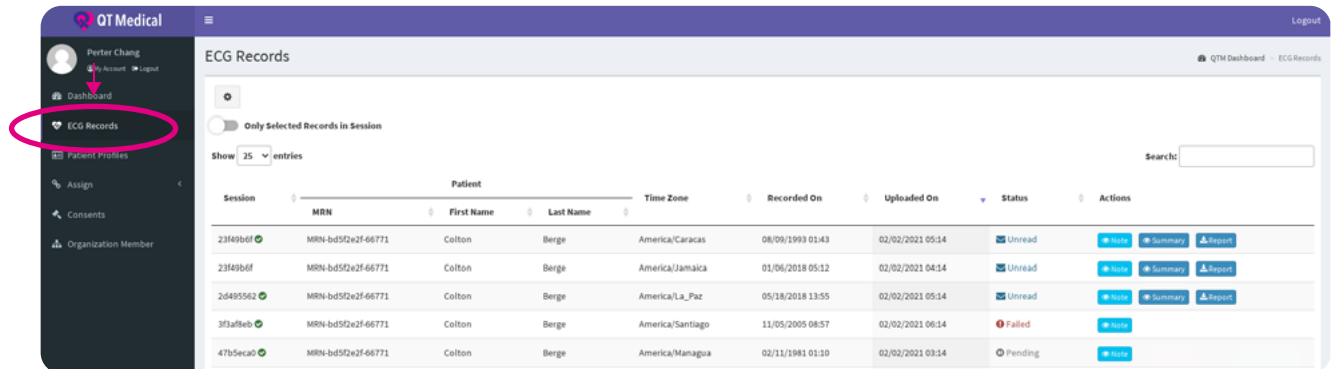
This is the correct record.

Session	MRN	Patient	Time Zone	Recorded On	Uploaded On	Status	Actions
		First Name	Last Name				
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Caracas	08/09/1993 01:43	02/02/2021 05:14	Unread Note Summary Report
2d495562	MRN-bd5f2e2f-66771	Colton	Berge	America/La_Paz	05/18/2018 13:55	02/02/2021 05:14	Unread Note Summary Report
3f3a9beb	MRN-bd5f2e2f-66771	Colton	Berge	America/Santiago	11/05/2005 08:57	02/02/2021 06:14	Failed Note
47b5eca0	MRN-bd5f2e2f-66771	Colton	Berge	America/Managua	02/11/1981 01:30	02/02/2021 03:14	Pending Note
6c2da029	MRN-bd5f2e2f-66771	Colton	Berge	Asia/Samarkand	04/02/2013 06:57	02/02/2021 14:14	Processing Note

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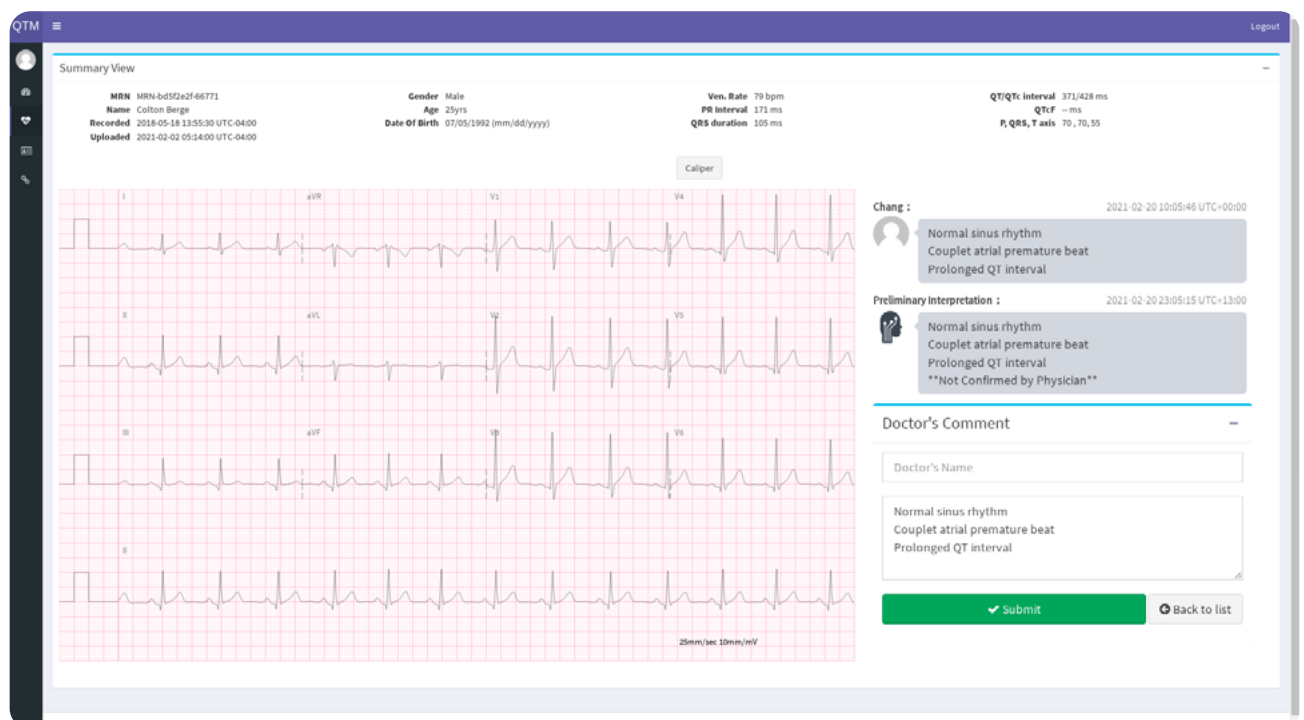
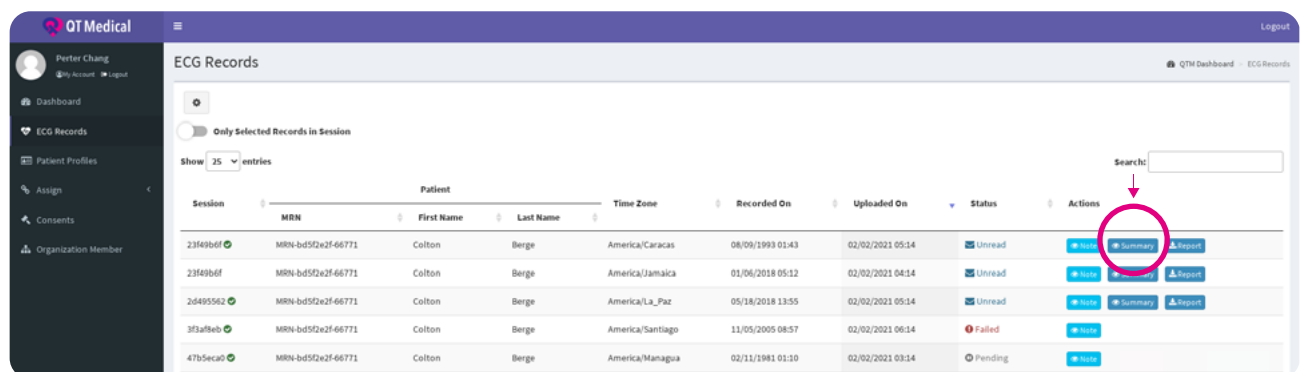
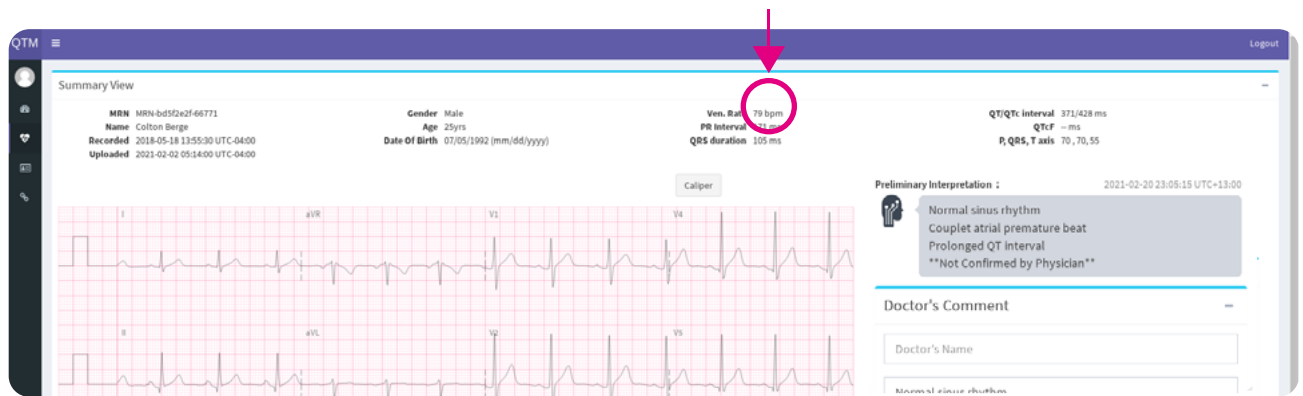
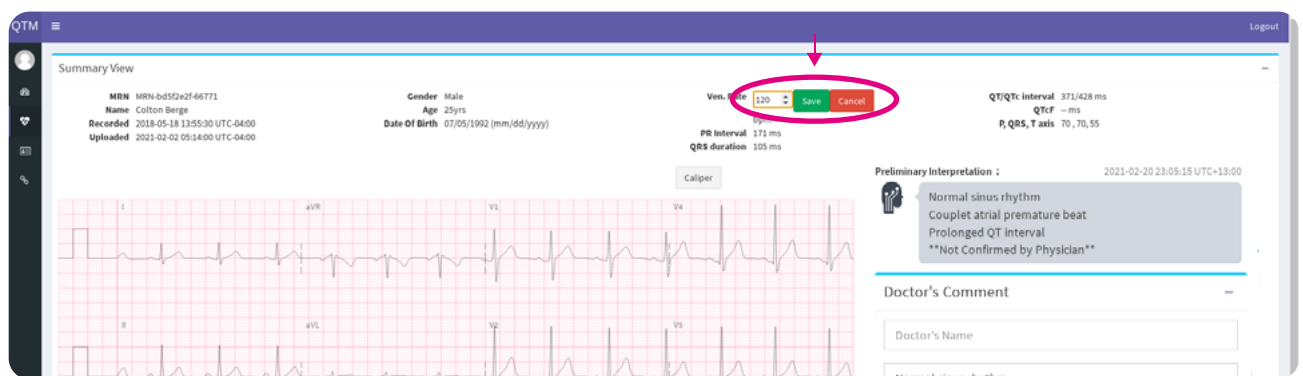
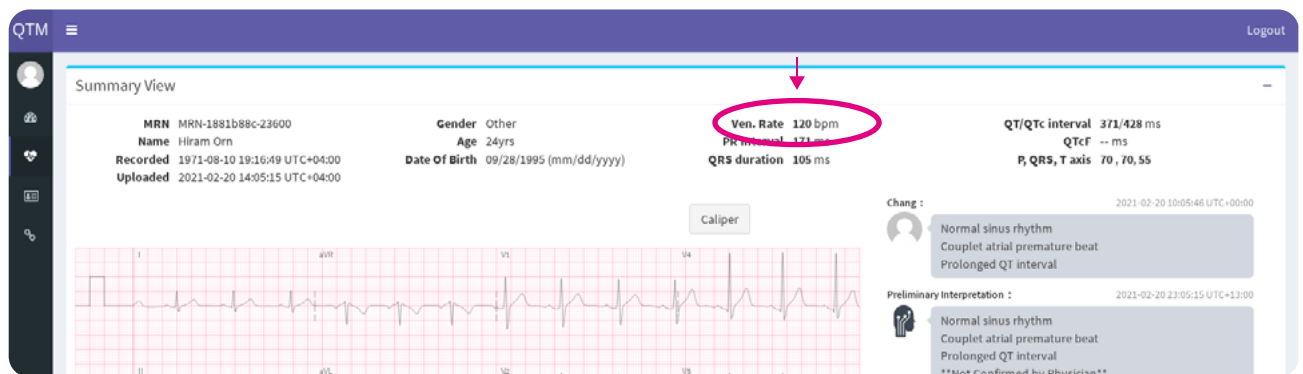
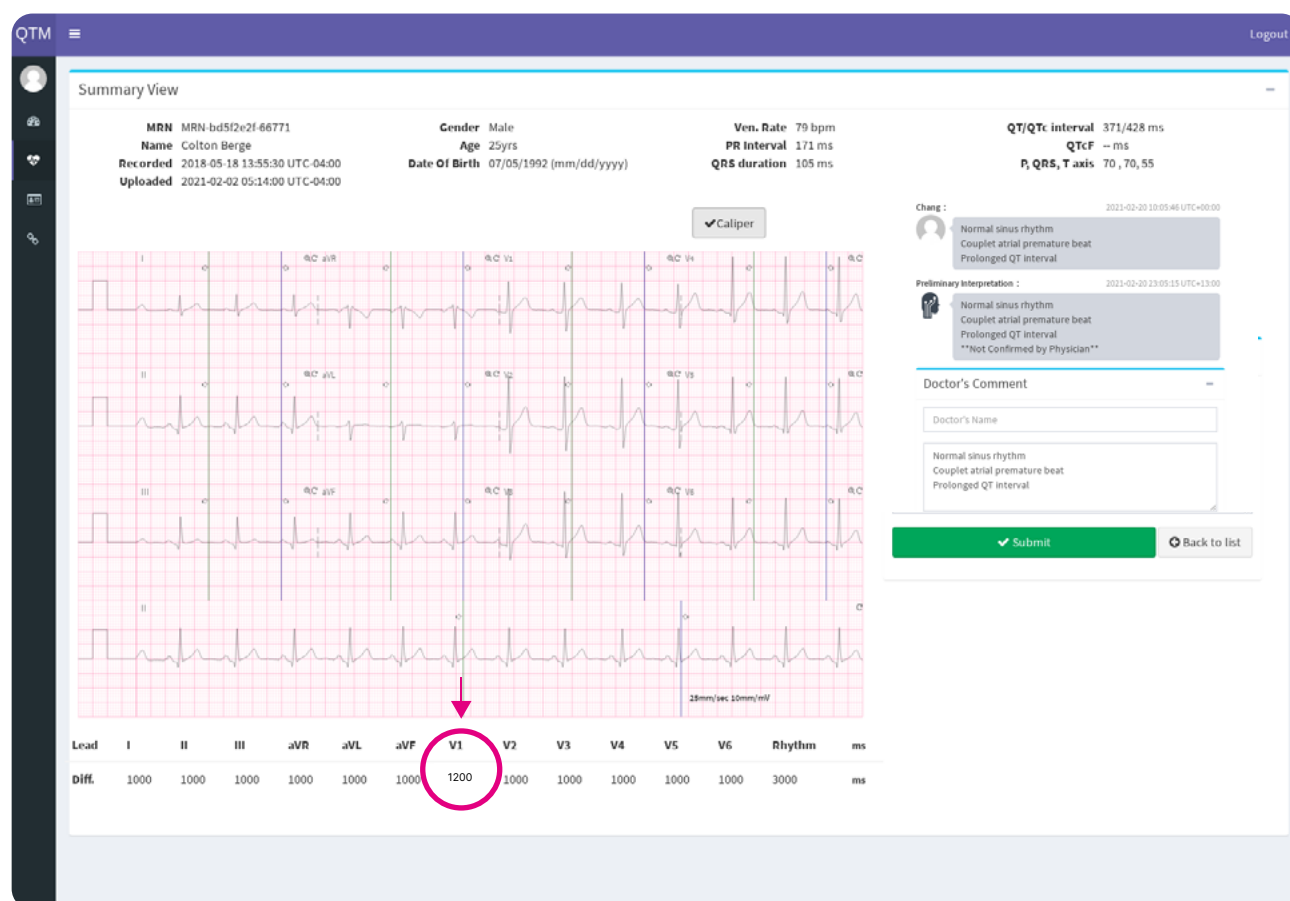
The screenshot shows the QTM ECG interface. At the top, there's a 'Summary View' header. Below it, patient information is displayed: MRN 1881b88c-23600, Name Hiram Orr, Gender Other, Age 24yrs, Recorded 1971-08-10 19:16:49 UTC+04:00, Date Of Birth 09/28/1995 (mm/dd/yyyy), Uploaded 2021-02-20 14:05:15 UTC+04:00. Clinical data includes Ven. Rate 120 bpm, PR interval 171 ms, QRS duration 105 ms, QT/QTc interval 371/428 ms, QTcF -- ms, P, QRS, T axis 70, 70, 55. A 'Caliper' button is highlighted with a red circle. The ECG waveforms are displayed in a 12-lead format. On the right, there's a 'Chang' section with a date and time, a 'Preliminary Interpretation' section with a date and time, and a 'Doctor's Comment' section with a text area and a 'Submit' button.

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The screenshot shows the QTM ECG interface with the 'Caliper' tool active. The patient information is: MRN bd5f2e2f-66771, Name Colton Berge, Gender Male, Age 25yrs, Recorded 2018-05-18 13:55:30 UTC-04:00, Date Of Birth 07/05/1992 (mm/dd/yyyy), Uploaded 2021-02-02 05:14:00 UTC-04:00. Clinical data includes Ven. Rate 120 bpm, PR interval 171 ms, QRS duration 105 ms, QT/QTc interval 371/428 ms, QTcF -- ms, P, QRS, T axis 70, 70, 55. The 'Caliper' button is now checked. The ECG waveforms are displayed in a 12-lead format. On the right, there's a 'Chang' section with a date and time, a 'Preliminary Interpretation' section with a date and time, and a 'Doctor's Comment' section with a text area and a 'Submit' button. A red circle highlights the 'Caliper' button, and another red circle highlights the 'V1' lead in the bottom table.

Lead	I	II	III	aVR	aVL	aVF	V1	V2	V3	V4	V5	V6	Rhythm	ms
Diff.	1000	1000	1000	1000	1000	1000	1000	1000	1000	1000	1000	1000	3000	ms

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The screenshot displays the 'Summary View' of an ECG report. At the top, patient information is shown: MRN (bd5f2e2f-66771), Name (Colton Berge), Gender (Male), Age (25yrs), Date Of Birth (07/05/1992), Ven. Rate (79 bpm), PR Interval (171 ms), QRS duration (105 ms), QT/QTc interval (371/428 ms), QTcF (ms), and P, QRS, T axis (70, 70, 55). The ECG traces are displayed in a grid format. A 'Caliper' tool is visible above the traces. On the right side, a 'Preliminary Interpretation' box is highlighted with a red border, containing the text: 'Normal sinus rhythm', 'Couplet atrial premature beat', 'Prolonged QT interval', and '**Not Confirmed by Physician**'. Below this, the 'Doctor's Comment' section is visible, with a text area containing the same interpretation text and a 'Submit' button.

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This screenshot shows the same ECG Summary View as the previous one, but with the 'Doctor's Comment' section highlighted by a red border. The 'Preliminary Interpretation' box remains visible above it. The 'Doctor's Comment' section includes a text area where the doctor can enter their diagnosis and notes, and a 'Submit' button at the bottom.

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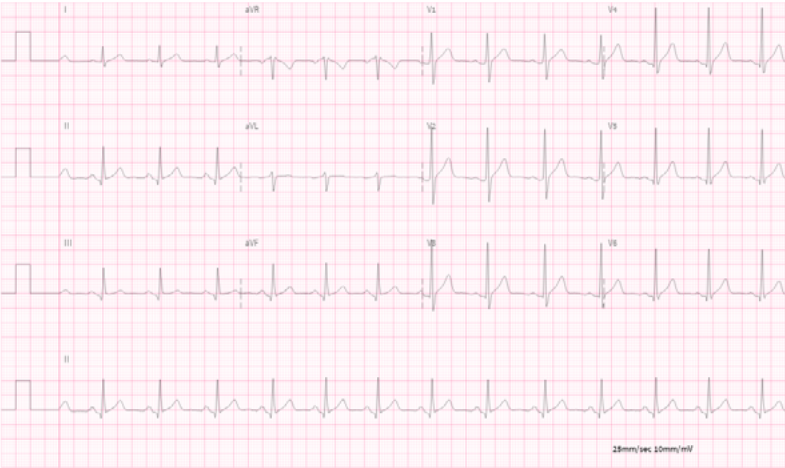
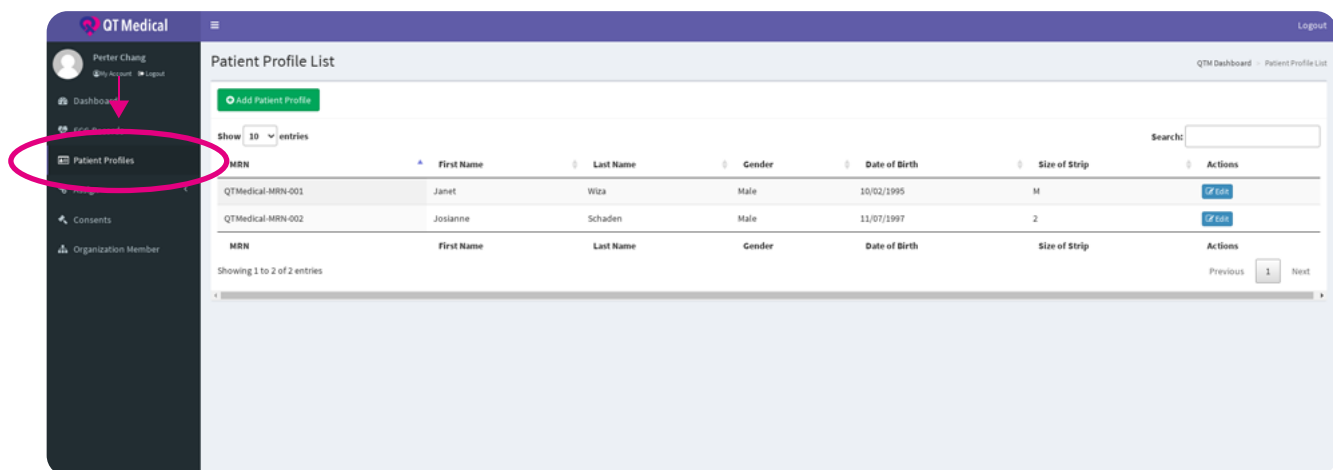
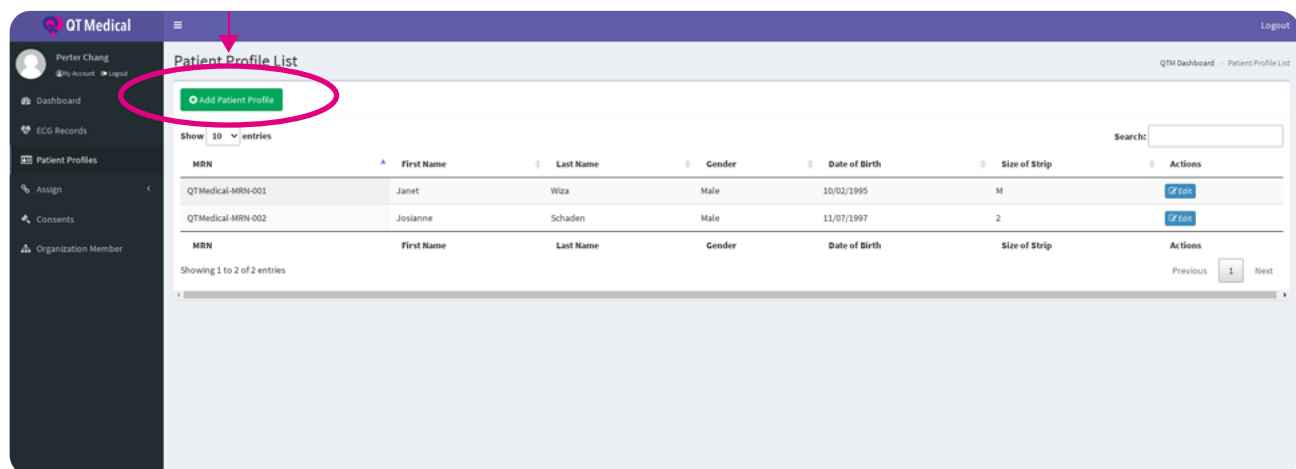
QT Medical

Perter Chang
My Account Logout

Dashboard
ECG Records
Patient Profiles
Assign
Consents
Organization Member

Create Patient Profile

QTM Dashboard Patient Profile List Create Patient Profile

* MRN
MRN

* First Name
First Name

* Last Name
Last Name

* Gender
Please choose your gender

* Date of Birthday
mm/dd/yyyy

* Size of Strip
Please choose your size

Create

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QT Medical

Perter Chang
My Account Logout

Dashboard
ECG Records
Patient Profiles
Assign
Consents
Organization Member

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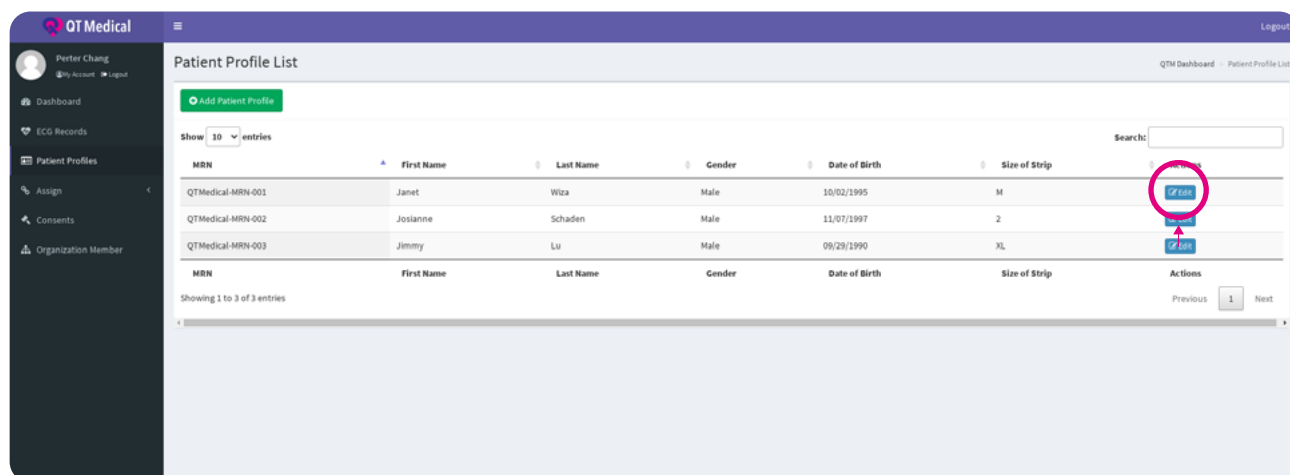
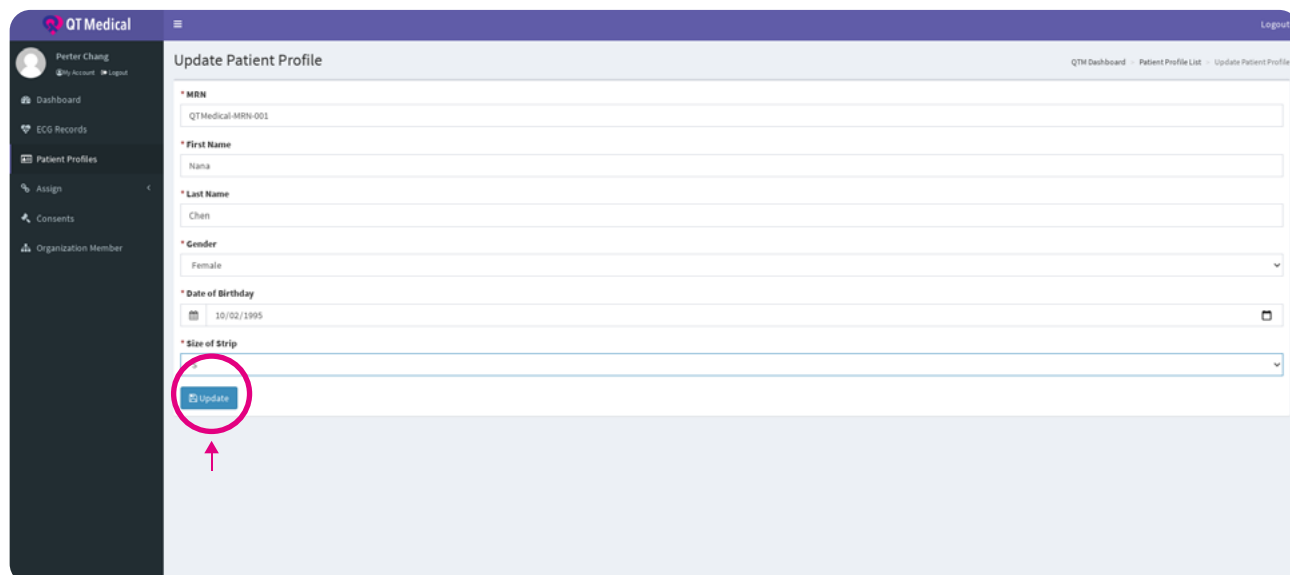
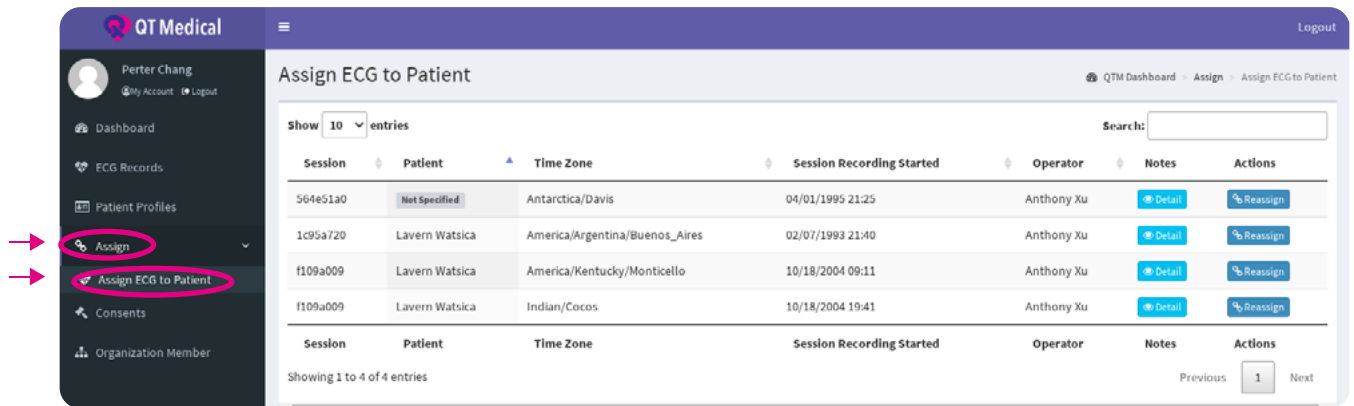
QT Medical

Assign ECG to Patient

QTM Dashboard > Assign > Assign ECG to Patient

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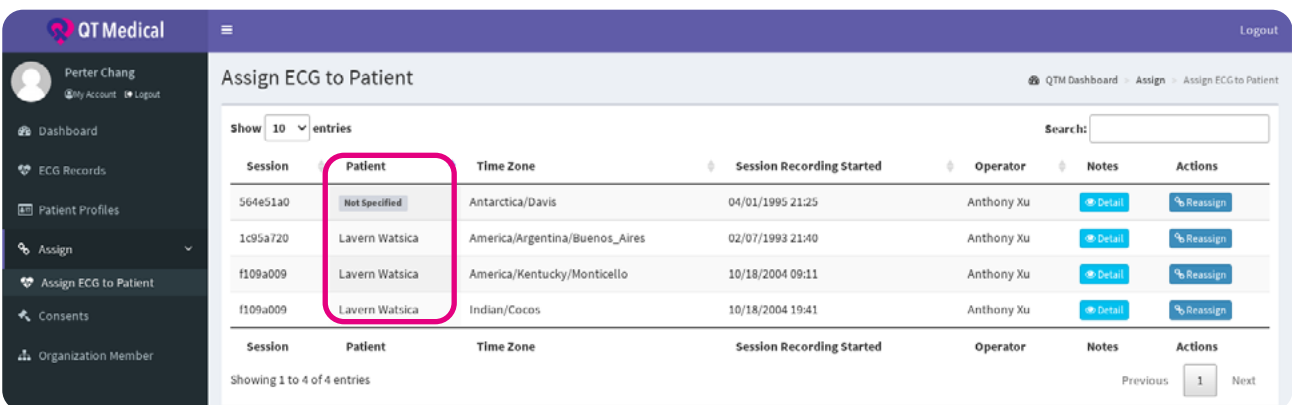
Show 10 entries

Search:

MRN	First Name	Last Name	Actions
MRN-5a108746-71232	Lavern	Watsica	Select

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Notes for ECG Session ID: f109a009

ECG #0 Note

The patient moves during recording.

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