

# QT ECG

12-Lead ECG

## Quick Guide


Version: V3-R2-01



For additional languages, please visit:  
<https://www.qtmedical.com/quick-guide>



Manufacturer: **QT Medical, Inc.**  
<https://www.qtmedical.com>

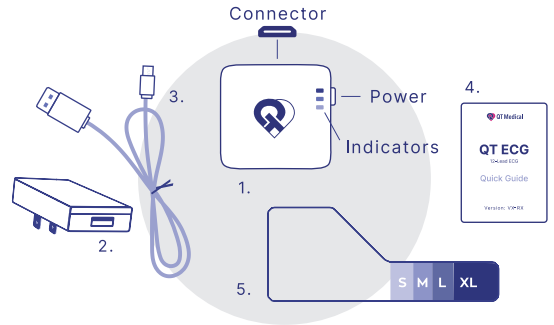

## Know Your Unit

1. ECG recorder
2. Wall Charger
3. Micro USB Cable
4. Quick Guide
5. Sizing Guide

**Warning:**  
Use only the wall charger and  
micro USB cable provided for charging.  
Failure to do so may damage the product.



### 1 Check Compatibility List

Make sure your device is on the  
compatibility list.

[www.qtmedical.com/compatibility](http://www.qtmedical.com/compatibility)

### 2 Install APP & Log In

Search: QT ECG  
Contact your local service team or  
distributor to obtain your account  
details




### 3 Find & Read User Manual Before Use

Find more information in APP or go to <https://www.qtmedical.com/support>



### 4 Long Press to Power on

It is recommended to charge the device every 30 days.

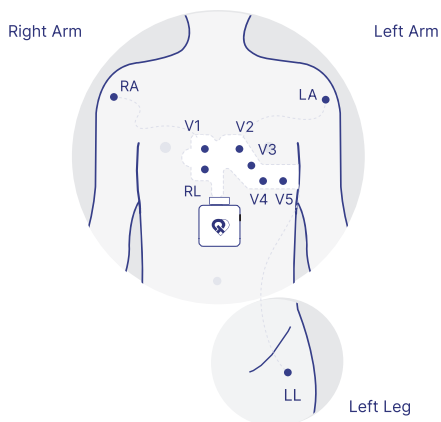

### 5 Refer to Accessories Label for Instructions

## Know Your LEDs

STATUS	Off	Blinking	solid
G:Green	B:Blue	A:Amber	Y:Yellow
R:Red			

1 Power on	2 Connected	3 Recording
G PWR	G PWR	G PWR
B CON	B CON	B CON
REC	REC	A REC

NOTICE	Low Battery	Charging	Error
Y PWR	Y PWR	R PWR	